



SET UP GUIDE

LineTab Waterline Maintenance Tablets are formulated for use in dental unit waterlines. Designed for continuous use, LineTab helps keep your waterlines clean and compliant.

Before Starting with LineTab™

First shock your waterlines using diluted bleach or a shock product recommended by your equipment manufacturer to remove any existing biofilm buildup. Repeat the shock procedure until all waterlines tests at 0 CFUs. For detailed instructions on how to shock your lines, **visit: linetab.us/how-to-use**.

Directions for Use

1 Test waterlines, make sure they test at **0 CFUs**.

2 Fill bottle with tap / distilled water, then add LineTab.

- For bottles sized **0.7 - 1L use 1 tablet**.
- For bottles sized **1.5 - 2L use 2 tablets**.

3 Repeat step 2 whenever refilling dental water bottles. It is not necessary to purge lines at the end of each day.

PRO TIP: Many dental offices prepare a premixed gallon of LineTab and water solution to conveniently fill their bottles (**4 tablets to one gallon of water**.) This method ensures a consistent LineTab-to-water ratio and allows offices to top-up bottles.

Water testing verifies that compliance standards are being met and that your protocols are working effectively. We recommend testing your water quality every quarter to ensure your waterline maintenance program is performing as intended. If results near or exceed the EPA/CDC limit of 500 CFUs/mL, perform a shock treatment and retest. **However, we recommend shocking your lines when they test above 0 CFUs.**